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# Instruction for Use

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EN

Latest-generation transfer boards

Please read the Instruction for Use carefully before first use!

### Icon legend

	Keep dry		Max. weight load		Observe the Instruction for Use
	Warning		Manufacturer		Mark of conformity
	Temperature limit		Keep away from sunlight		Medical device designation
	Item number		Batch no./Lot no.		Slip hazard

## 1. Technical data

REF	Item designation / variants	Dimensions	Medical aid no.	UDI-DI
4500414	Transa Silver Glider M	67 x 22 x 1 cm	22.29.01.3017	(01)04050856041463(11)xx(10)xx
4500415	Transa Silver Glider L	85 x 25 x 1.2 cm	22.29.01.3018	(01)04050856041470(11)xx(10)xx

Material	Colour	Weight load
HDPE (high-density polyethylene)	silver	130 kg with <30 cm clearance 180 kg with <20 cm clearance

### 1.1. Storage and transport

Storage temperature: 5° to 25°

Storage conditions: keep dry, protect from moisture and direct sunlight

Humidity: no information

1.2. Scope of delivery: Transa Silver Glider, Instructions for Use

### 1.3. Accessories

Item No.	Item designation	Dimensions	Material	Colour	UDI-DI
4410514	Anti-slip pad	180 x 90 mm	Cellular rubber/CR/ EPDM-L70	black	
4500418	Anti-slip pad for SilverGlider M	145 x 180 mm	TPU (thermoplastic polyurethane)	transparent	
4500419	Anti-slip pad for Silver Glider L	200 x 200 mm	TPU (thermoplastic polyurethane)	transparent	
4410525	Transa Sit-On Cloth	305 x 580 mm	Polyester with PVC coating	blue	(01)04050856043320 (11)xx(10)xx

## 2. Indication / application areas / intended use

No specific clinical picture can be given as an indication because of the various possible applications. The Transa Silver Glider is an extremely solid and compact transfer aid made of polyethylene. When used and maintained properly, the surface of the Transa Silver Glider permanently retains its sliding properties. The compact dimensions allow the caregiver to move a patient from the edge of the bed to the wheelchair/chair, etc. and in particular, from the wheelchair to the car. The two large, transparent anti-slip pads beneath the two ends are used to provide stable support on both sides. A handle makes it easy to carry the Transa Silver Glider.

## 3. Multiple use

The Transa Silver Glider transfer aids are intended for multiple use. They are not a single-use product. It is essential to conduct a thorough functional check and to clean or disinfect the Transa Silber Glider before reuse.

## 4. Contraindications / risks of use

There are no known contraindications.

Users must be trained. If users are not trained or acquainted with the correct handling of the Transa Silver Glider, the Transa Silver Glider may be used incorrectly. Observe the usage instructions in Section 5 and the safety instructions in Section 7. It is the sole responsibility of the user to determine the suitability of the Transa Silver Glider for the specific application. Transatlantic assumes no responsibility for any improper use of Transatlantic transfer products.

---

## 5. Usage / placement

---

The Transa Silver Glider is typically used by trained nursing staff or a trained individual. The Transa Silver Glider can be used by the patient/user alone if they have received appropriate training, particularly for transfer from the wheelchair to the car.

- The Transa Silver Glider is positioned with one end underneath the patient/user and the other end securely placed on the other level.
- The transfer can take place in small steps or a smooth slide.
- Textile clothing should preferably be worn. With materials such as leather, increased slip resistance can cause restrictions.
- The Transa Silver Glider is removed after the transfer.

**IMPORTANT:** The Transa Silver Glider must be securely positioned before the transfer.

**This product is suitable for multiple use.**

---

## 6. Service life / life cycle

---

If the Transa Silver Glider is used according to the safety instructions, the expected service life is 5 years. There is no limitation to this period.

A longer or shorter service life is possible depending on the load. Usability may also end prematurely in the event of relevant damage to Transa Silver Glider.

---

## 7. Safety instructions / functional check

---

**WARNING** signifies a possible hazard that could lead to personal injury or material damage.

The Silver Glider is not suitable for any applications that deviate from the intended use and could result in hazards. No negative effects on the user or due to the use of the slider board have yet been identified. In this respect, no contraindications can be identified.

- **Slip hazard!** Ensure that the patient does not slide off the bed, the (wheel)chair, etc. during transfer or repositioning.
- **Risk of accident!** Always make sure when using the Transa Silver Glider that the bed, chair or any movable base is stable, i.e. that any brakes on it are locked.
- **Risk of breakage!** The gap between the aid and the surface must be kept as small as possible to ensure maximum safety. Care should also be taken to ensure that the distance between both transfer units does not exceed 30 cm.
- Defective products may no longer be used.
- It is the responsibility of the user/institution to evaluate if multiple users may be needed for the transfer or if additional safety measures must be taken.
- The maximum weight limit of 180 kg must be complied with.

Repositioning products are also subject to a natural aging process. Inspect the product regularly and carefully. Ensure that the product is in flawless condition and that the material has not been damaged.

Transatlantic transfer aids are subject to the highest quality requirements. Any serious incidents must be reported to the manufacturer and the competent authority in the Member State where the user and/or patient is based.

## 8. Cleaning, care and disinfection

The Transa Silver Glider must be cleaned prior to and after use with each change of patient.

The Transa Silver Glider itself is extremely durable and can be cleaned using normal household cleaners. When cleaning, do not use abrasive or bleaching agents and objects with sharp edges. If bleaching agents or cleaning agents containing alcohol are used, the functional property of the surface may be impaired long term.

The resistant material of the Transa Silver Glider can be treated using a standard wipe-down disinfection. Check the compatibility beforehand, using the information given on the disinfectant if applicable. We recommend agents that are free from aldehyde and alcohol such as Trionic (DGHM/VAH -approved) (Disinfectants Commission in the Association for Applied Hygiene).

The Transa Silver Glider is unsuitable for use in an autoclave due to the thermo-sensitive structure of the material.

## 9. Product marking /CE

The Transa Silver Glider is a Class I medical device and is compliant with EU Regulation 2017/745 of the European Parliament and of the Council dated 05/04/2017 regarding medical devices (MDR).

We are certified as a manufacturer and distributor of medical products in accordance with DIN EN ISO 9001 and DIN EN ISO 13485. Our medical products have, of course, undergone the relevant conformity assessment procedure and therefore bear the CE mark.

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## 10. Warranty

The warranty period is 2 years. This does not extend to defects that are attributable to improper use or normal wear and tear.

Transatlantic assumes no liability for accidental or indirect injury or for material damage.

## 11. Disposal information

There is no special disposal regulation for the Transa Silver Glider. It must be disposed of in accordance with national and local regulations.

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